

SONIC FFP2 MASK TECHNICAL DATA SHEET

FLEXI-JUST HEAD HARNESS



With the flexible head harness sitting below the ear and over the crown of your head, this makes for an easy and comfortable fit. It also allows for the mask to be adjusted to a perfect fit on the contours of all faces

REFINE-AIR



We have taken steps to ensure the highest protection factors for our mask users. With our high-efficiency filter medium we ensure best lung protection, low breathing resistance, comfortable fit and performance under pressure.

APPROVALS

- SAHPRA Approved
- SABS Approved
- NRCS
- Sodium Chloride test passed/cleared
- Paraffin oil test passed/cleared
- Dolomite dust resistant
- Blood spatter resistant

INTERNAL NOSE CLIP



We have fitted our FFP's with an enclosed nose clip, this prevents any accidents and ARC's in ZONE-0 environments

NO LEAK GUARANTEE



With our ultrasonic-seal there is no cut, burn or glue. This means that there will be no leaking on any seam or fold.

COOL-360° EXHALATION VALVE



The Exhalation Valve has been fitted to optional FFP's for the ease of breathing/exhalation. The valve has proved to have a relieve of the heat build-up inside the mask while used. The downward facing valve also allows the exhaled breath to blow down and not into the face of the next person.



SONIC PROTECTIVE GEAR DISPOSABLE RESPIRATORS

The PSA Sonic Mask FFP2 Flat Fold with active carbon is a disposable respiratory mask created for settings with moderate levels of fine dust and non-oil-based particles. It is frequently employed in industries like construction, welding, pharmaceuticals, and agriculture. Its main purpose is to protect the wearer from harmful particles by minimizing exposure. The mask includes a soft, cushioned lining for lasting comfort during work, a Flexi-Just Head Harness, an internal nose clip, soft edges, and nose foam to fit various face shapes.

APPROVALS AND STANDARDS:

- ▶ NRCS Homologation
- ▶ SANS 50149:2003 NR
- ▶ SAHPRA Approved
- ▶ SAB Local content
- ▶ ARMSCOR Tested

THE FOLLOWING MATERIALS ARE USED IN THE PRODUCTION OF THE SONIC FLATFOLD FFP2:

HEADSTRAP	Woven Textile with rubber elastic inner
NOSE CLIP	Aluminium enclosed within FFP2
EXHALATION VALVE BODY	Polypropylene
EXHALATION VALVE DISC	Rubber

USES

USE FOR

- ▶ Intended to be worn by operating room personnel during surgical procedures to help protect both the surgical patient and the operating room personnel from transfer of micro-organisms, body fluids, and particulate material.
- ▶ Intended for use in industrial environments.
- ▶ Always follow user instructions and use in manners as indicated.

DO NOT USE FOR

- ▶ DO NOT use for gases or vapours (i.e. anesthetic gases such as isoflurane or vapours from sterilants such as glutaraldehyde.)
- ▶ DO NOT use in any manner not indicated in the user instructions

FITTING INSTRUCTIONS

Must be followed each time the respirator is worn. Before fitting device, ensure hands are clean. All respirator components should be inspected for damage prior to each use.

1. With reverse side up and using the tab, separate top and bottom panels to form a cup shape. Bend slightly at centre of the nose clip.
2. Ensure both panels are fully unfolded.
3. Cup respirator in one hand with open side towards face. Take both straps in other hand. Hold respirator under chin, with nose piece up, and pull straps over head.
4. Locate the upper strap across the crown of the head and the lower strap below the ears. Straps must not be twisted. Adjust top and bottom panels for comfortable fit, ensuring panels and chin tab are not folded in.
5. Using both hands, mould nose clip to the shape of the lower part of the nose to ensure a close fit and good seal. Pinching the nose clip using only one hand may result in less effective respirator performance – use two hands.
6. The seal of the respirator on the face should be fit checked before entering the workplace.

STORAGE AND TRANSPORTATION:

Shelf life of the unopened product is three (3) years from date of manufacture when stored within temperature range of 20°C to 30°C and at less than 80% relative humidity. End of shelf life date is marked on the product packaging. Before initial use, always check that the product is within the stated shelf life. When storing or transporting this product use original packaging provided.

GENERAL DATA

Manufacturer	Sonic FFP2 Mask, manufactured by Zodiac Manufacturing (Pty) Ltd
Designation	FFP2 Particle Filtering Mask - Sonic Protective Gear
Intended use	Protection during surgical procedures to help protect both the surgical patient and the operating room personnel from transfer of micro-organisms, body fluids, and particulate material. Intended for use in industrial environments and mining sector.
Standards	SANS 50149:2003 Dust clogging protection SANS 1866-2:2018 Blood spatter protection
Approval	NRCS - National Regulator for Compulsory Specifications SAHPRA Approved, SABS Approved, ARMSCOR Tested

DESIGN

Material	Headband: Woven Textile with rubber elastic inner Nose clip: Aluminium enclosed within FFP2
Model principle	Sonic FFP2 Mask ensures a high-efficiency filter medium, best lung protection and low breathing resistance. The ultrasonic-seal ensures no leaking on any seam or fold
Shelf live	Shelf life of the unopened product is three (3) years from date of manufacture when stored within temperature range of 20°C to 30°C and at less than 80% relative humidity.

PACKAGING

Packing	Label: Sonic Protective Gear FFP2 Mask. Markings in accordance with EN 149:2001 + A1:2009 SANS 50149:2003, expiry date. Packaging contains safety instructions and instructions to use
Units per Package	20pcs per box

SONIC FFP2 WITH ACTIVE CARBON

PSA

WHERE SAFETY MEETS TECHNOLOGY

LAYERED



Consists of 6 layers, non-woven, high electrostatic charged filtering media to protect against solid and liquid non-volatile particles.

REFINE-AIR



High-efficiency filter medium we ensure best lung protection, low breathing resistance, comfortable fit and performance under pressure.

CARBON ACTIVE



Protection against hazardous and harmful, vapour and smoke particles that are not radioactive, nor airborne biological substances from risk group 3, nor enzymes

APPROVALS



- SAHPRA Approved
- SABS Approved
- NRCS
- Sodium Chloride test passed/cleared
- Paraffin oil test passed/cleared

COOL-360° EXHALATION VALVE




The Exhalation Valve has been fitted to optional FFP's for the ease of breathing/exhalation. The valve has proved to have a relieve of the heat build-up inside the mask while used. The downward facing valve also allows the exhaled breath to blow down and not into the face of the next person.

NO LEAK GUARANTEE



With our ultrasonic-seal there is no cut, burn or glue. This means that there will be no leaking on any seam or fold.

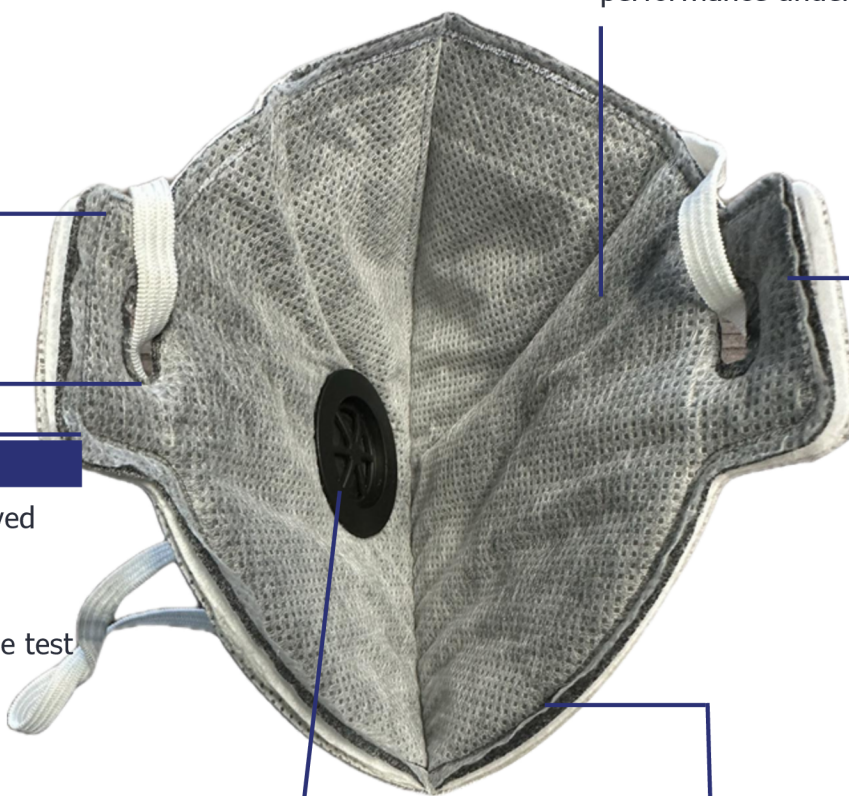

AREAS OF APPLICATION		
ACTIVITY	HARMFUL SUBSTANCES	PROTECTION CLASS
Wood Processing		
Soft Wood: cutting & sanding	fine particles, wood dust	FFP2
Paint: brushing& sanding	fine colour particles	FFP2
Construction and Ancillary Building Industry		
Cement: working, plastering, tiling, roofing	cement/plaster/tile/brick dust	FFP2
Masonry: cutting, drilling, demolition	cement/plaster/tile/brick dust	FFP2
Roof Insulation: glass/mineral fibers	dust & fibers	FFP2
Body Fillers/Stopppers: grinding	dust	FFP2
Welding		
Welding: aluminium & stainless steel	metal/oxide fumes	FFP2
Brazing	fumes	FFP2
Metalwork		
High-alloy Steels: drilling, grinding, milling, angle grinding	metal dust	FFP2
Stainless Steels: drilling, grinding, milling, angle grinding	metal dust	FFP2
Waste Disposal and Cleaning		
Refuse: separation and disposal	dust/fungi	FFP2
Medicine and Healthcare		
Allergies, Pollen, Household Dust, Animal Fur	dust/particles/spores	FFP2

PERFORMANCE				
Min. DATA IN ACCORDANCE WITH STANDARD, INCL LOADING TEST WITH 120 MG PARAFIN OIL.				
Particle performance	Standardised Testing	0,3 Micron	FFP2: EN149:2001	FFP2: Sonic
	Test aerosols:			
	Min. Efficiency	Sodium Chloride	94%	99%
		Paraffin oil:	94%	99%
Breathing resistance	Inhalation	30 litres/min	Max.0,7	0,6
	Inhalation	95 litres/min	Max.2,4	2,1



*Gas Detection Instrumentation, Respiratory
Protective Equipment, Alcohol Breathalysers
and Specialised Safety Equipment*

Contact



enquiries@psaafrica.co.za



0860 543 356



www.psaafrica.co.za